HEINE QUALITY
MADE IN GERMANY

∴ HEINE mPack LL®

CE 93/42/EWG/CEE/EEC

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com
med 0209 1/12.10

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Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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Norme applicate**ISO 7000: 2004**

(Graphical symbols for use on equipment)

EN 980:2008

(Graphical symbols for use on equipment)

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IEC 60601-1: 1988 + A1:91 + A2:95 (2nd Edition)

(Medical Electrical Equipment – General Requirements for Safety)

UL 60601-1: 2003

(General requirements for Safety for Medical Electrical Equipment)

Instructions for use

Read these instructions carefully before using the HEINE mPack LL® and keep them in a safe place for future reference.

Safety Information

Range of applications: The HEINE mPack LL® power source is designed solely for use with HEINE LED LoupeLight®. When wall-mounted, the instrument support is designed for use with HEINE headbands and S-Frames only.

General Safety: All repairs should be carried out by qualified personnel, with the exception of a battery change with mPack LL. When changing the battery, use only the original rechargeable battery HEINE Li-ion mPack LL®.

Safety in use:

- Follow the instructions supplied with the unit regarding installation of the instrument support and the wall-mounting bracket.
- For wall-mounting, be sure to use the instrument support (1) supplied with the unit to be sure that the instrument is located safely. Always use the instrument support (8) for the S-Frame. Be sure to attach the S-Frame securely to the instrument support as shown in (9).
- Make sure that the local mains voltage is the same as that shown on the identification plate of the plug-in transformer.
- Use only the appropriate HEINE plug-in transformer to connect the unit to the local mains supply.
- If not used for long periods, unplug the mains transformer from the mains.
- The ideal temperature for charging is between +10° and +35 °C.
- Do not use the HEINE mPack LL® in rooms where an explosion hazard is present.
- We can only guarantee the safety and performance of this unit if it is fitted with original HEINE accessories and spare parts.
- When moving from a cold to a warm environment, leave the power source switched off for up to one hour to allow it to adapt to the new conditions.
- The mPack LL has been tested for electro-magnetic compatibility and radiation. The unit is approved for use in a medical environment.

Symbols

In accordance with local regulations this product should be disposed of as an electronic device separately.

Double insulated equipment of Protection Class II

Observe the safety information.

WITH RESPECT TO ELECTRICAL SHOCK FIRE, AND MECHANICAL HAZARDS ONLY IN ACCORDANCE WITH UL60601-1 & CAN/CSA-C22.2 No. 601.1-M90

IP40

Protection class: Protected from the ingress of small objects, not water-resistant.

Setting up and operation

1 HEINE mPack LL®

- Charge the mPack LL for two hours before using it for the first time. Charge the mPack LL via the connector (11) with a separate HEINE mains transformer or in the charging socket of the base unit (7).
- Connect the instrument to the connector (2) and select the required brightness by means of the brightness control (3). The HEINE mPack LL® switches off with an audible click after use.
- With the mPack and base unit (7) it is possible to use a HEINE instrument under static or mobile conditions if preferred. The mPack will be recharged automatically when the attached HEINE instrument is switched off.
- The indicator (4) lights up as soon as the mPack LL is switched on at (3). It must switch off when the mPack LL is switched off.
- General conditions for charging:
 - Recharging time depends on the state of charge of the battery and lasts for a maximum of 2 hours.
 - When the battery is fully-charged the mPack LL charging logic switches off, all LEDs are switched off.
 - The battery cannot be overcharged.

2 HEINE mPack LL® for wall-mounting

- Attach the base unit to the wall by means of the wall bracket (6). Slide the instrument support (1) or (8) onto the hinged support (5). Use (1) for the headband or (8) for the S-Frame.
- Inserting the S-Frame into the instrument support:
 - Insert the S-Frame under the holder (9).
 - Make sure the S-Frame is correctly-positioned as in (10) so that it is properly retained in position.
- The indicator (4) must light up as soon as the mPack LL is switched on at (3). It must go out as soon as the mPack LL is switched off or the hinged support is pressed down. When the instrument is removed the mPack LL switches on the instrument automatically.
- The mPack LL can be recharged whether the power source is switched on or not or whether an instrument is connected or not.
- The LED indicator works independently whether the mPack is used with the base unit or not (see LED indicator).

3 Direct Use

If the battery of the mPack LL is completely discharged you can continue to use the instrument with the mains transformer or in the EN 50.

Important note: During direct use with the mains transformer or in the EN50, the battery will not be recharged simultaneously.

4 LED indicators (4) on the HEINE mPack LL®

- During charging the LED's (4) operate sequentially, in use they remain switched on.
- If the LEDs (4) remain switched off, the unit is non-operational or the mPack LL is switched off (e.g. If the battery is completely discharged).
- The number of illuminated or sequentially-switched LEDs (4) indicates the state of charge of the Li-ion battery. The state of charge is shown whether an instrument is connected or not.
- All five LEDs illuminated indicate a fully-charged battery.
- If the yellow LED flashes on and off, the battery must be recharged. Switch off the mPack LL and recharge it by means of the HEINE mains transformer or the base unit (7).

Important note: The mPack LL switches off automatically (LED's switched off) if the instrument used is defective.

5 Changing the battery

The Li-ion mPack LL only needs replacement when the battery no longer recharges to give sufficient capacity. This usually occurs after 3 to 5 years if properly used.

- Disconnect the plug-in transformer of the mPack LL from the mains or remove the mPack from the base unit (7).
- Unscrew the lid screw on the back of the mPack LL and remove the lid.
- Remove the old battery and disconnect the cord. No tools are necessary.
- Insert the new Li-ion mPack LL battery into the mPack so that the label is visible and the battery cord is directed to the front control (3).
- Connect the plug of the battery to the printed circuit board.
- Take care when closing the lid that the cord is not pinched between lid and body.
- To close, locate the two pins on the lid in the cut-outs in the lower half of the housing. Close the lid and tighten the screw.
- Charge the new Li-ion mPack LL battery for two hours before use.

Note: The HEINE Li-ion mPack LL® battery must be disposed of properly.

Care and maintenance

- Always disconnect from the mains before cleaning.
- We recommend that external surfaces of the mPack LL be wiped clean with a disinfectant cleaning wipe. This should only be carried out by qualified personnel.
- Any disinfectant used must be approved by the disinfectant manufacturer for use on plastics. The manufacturer's instructions must be followed.
- The external surfaces may also be wiped clean with a cloth moistened with water or with a neutral pH or slightly alkaline cleansing agent. Isopropyl alcohol may only be used at concentrations up to 70%.
- Spray disinfection and soaking are not permitted and may cause damage.
- The article can be neither machine-cleaned, nor disinfected at high temperature or sterilised. These processes lead to irreparable damage to the product.
- Correct operation of the device must be tested at appropriate intervals with regard to operating conditions.
- Use the units only when they are functioning correctly.

Specifications

HEINE base unit:

Mains voltage:	100–240 V~ / 50–60 Hz
Safety class:	Double insulated
Current consumption:	max. 30 VA
Output:	9 V DC max. 3 A
Fuse:	Integral overload protection
Duration of use:	Continuous
Length of cord:	200 cm

HEINE mPack LL®:

Output:	Variable, max. 7,5 W
Input:	9 V DC, max. 3 A DC
Fuse:	Integral overload protection
Charging current limit:	3 A
Weight:	ca. 300 g
Dimensions:	80 mm x 28 mm x 125 mm

Rechargeable Battery HEINE Li-ion mPack LL®:

Ambient conditions:	In use: +10 °C to +40 °C Storage: +5 °C to +45 °C Transport: -20 °C to +45 °C
Relative humidity:	30% to 75%
Air pressure:	700 hPa to 1060 hPa
Cell type:	Li-ion
Nominal voltage:	7,2 V
Charging current:	1-3 A, depending on the light source, conditions of use and state of charge of the battery. integrated overload protection
Fuse:	
Weight:	ca. 185 g
Dimensions:	70 mm x 73 mm x 19 mm

Applicable Standards**ISO 7000: 2004**

(Graphic symbols for use on equipment)

EN 980:2008

(Graphic symbols for use on equipment)

IEC 60601-1: 1988+A1:91 + A2:95 (2nd Edition)

(Medical Electrical Equipment – General Requirements for Safety)

UL 60601-1: 2003

(General requirements for Safety for Medical Electrical Equipment)

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WITH RESPECT TO ELECTRICAL SHOCK FIRE, AND MECHANICAL HAZARDS ONLY IN ACCORDANCE WITH UL60601-1/CAN/CSA C22.2 No. 601.1

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